
HUMANTECHNIK

lisa
sign●lux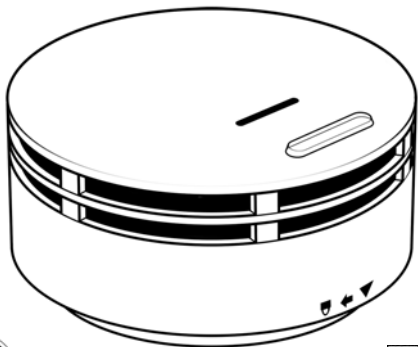

REF A-2638-0
A-2642-0
A-2672-0
A-2674-0

HUMANTECHNIK

lisa
sign●lux

DE

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EN

Additional information

FR

Notes complémentaires

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The *Lisa-Signolux* hazard detection devices help people with hearing problems to compensate for their disability. They enable the perception of dangers in living quarters which are caused by combustion, by enabling visible alert signals.

These detector devices are part of a medical system and are compatible with all devices in the *Lisa-Signolux* alerting system.

These additional instructions contain important information about the operation of your hazard detection device. Read these instructions in full before installing and commissioning your hazard detection device and keep them in a safe place.

Immediate safety information

- Read this additional information booklet carefully and completely before operating and using the device. Observe the other warnings and safety instructions in the main user manual.
- Keep the main user manual and this additional information in a safe place, so that they are accessible to other users at all times. Every main user must have access to them.
- Keep *Lisa-Signolux* devices out of the reach of children! This package contains small parts - there is a risk of choking and suffocation!

- The flashes of light emitted by the receiver devices can trigger epileptic seizures in people with photosensitive epilepsy (PSE).

Automatic pairing

Lisa-Signolux-devices can be used together immediately and without prior pairing when delivered.

When triggered by a hazard, the *Lisa-Signolux* detector devices are indicated by the receivers with the fire alarm symbol  by default.

When a function test is carried out, the alarm symbol  is triggered for differentiation.

Important notice 1:

If the detector is to be displayed with an event icon of your own choice, this can be done as described in the user manual.

However, this causes the according receiver to lose the automatic pairing function and every transmitter previously indicated on the receiver must then be individually paired.

Important notice 2:

Should there be repeated false alerts when using the Preset transmitter indication, this may hint at a neighbouring signalux installation causing interference. In this case, the problem can be solved by individually pairing all transmitter and receiver units within the setup.

To restore the automatic pairing function, please refer to the user manual of the receiver device.

Warnings and safety instructions

This chapter lists further important information for safe handling and other conditions for operation.

Safety instructions for batteries

The hazard detector devices contain long-life lithium batteries. These cannot be replaced.

Batteries must not be exposed to high or low extreme temperatures and low air pressure during use, storage or transport at high altitude.

Disposing of the device or the batteries in fires, ovens or by mechanical crushing and cutting may cause fires or explosions. Exposing batteries to extremely high temperatures may cause an explosion or leakage of flammable liquid or gas.

Batteries exposed to extremely low air pressure may explode or emit flammable liquid or gas.

Batteries are toxic and must not be swallowed! Keep them out of the reach of children and pets. In case of emergency, seek appropriate medical attention immediately.

If the battery charge level is insufficient, vital event messages from the system may be missed!

Information on product safety

Please observe these instructions to avoid damage to the unit and possible invalidation of the warranty.

This hazard detection device is designed for use in living areas only. The unit must not be exposed to direct sunlight for long periods of time and must also be protected from excessive heat, moisture or strong mechanical shocks.

The device is not protected against splash water.

Do not drop the unit. Dropping the unit on a hard surface can damage or destroy it.

Do not disassemble the device. Opening and modifying the unit will immediately void the warranty. Only use accessories from Humanteknik.

This device is not suitable for outdoor use. It must not get damp or wet.

Only use units of the *Lisa-Signolux* system where electronic devices are permitted. When using in aircraft, it is essential to obtain the prior consent of the flight personnel.

Strong interference signals from third-party devices can impair or completely prevent the function. Do not place potentially interfering devices such as screens and mobile phones closer than 30 cm to *Lisa-Signolux* devices and, if necessary, their cabling, as important event messages such as fire warnings may be prevented.

For your safety:

To ensure the function of the system, a function test must be carried out after installation. If smoke detectors are integrated into the system, it is a legal requirement to check the detectors and their alerting function regularly!

Environmental conditions

Units of the *Lisa-Signolux* system should only be operated in a dry environment within the guideline values for temperature and humidity specified in these operating instructions. Humidity and moisture can lead to failure of the units.

Markings

- MD** This is the labelling of Humantechnik GmbH for a medical product according to EU 2017/745.
- UDI** This is the marking for the product identification number to identify the medical device.
- SN** This is the marking for the serial number to identify the medical device. This is indicated on the device and the packaging.
- REF** This is the identification for the manufacturer's item number.

This is the marking for manufacturer of the medical device.

This symbol indicates the production date of the medical device.

This is the marking of the country of production of the medical device. This is shown on the device and the packaging.

This symbol indicates distributors for the medical device.

This symbol indicates that the user should read these operating instructions attentively.

This sign indicates warnings that are important for the user as well as important information on use and operational safety.

Temperature range for transport and storage: -5°C to 40°C

Temperature range for operation: 10°C to 30°C, i optimum ambient temperature: 25°C

Humidity during transport and storage: < 90% (non-condensing)

Operating humidity: 15% to 90% (non-condensing)

Atmospheric pressure range during transport, storage and operation: 700 hpa to 1060 hpa

This mark indicates that this appliance does not require sterilisation.

The CE mark on the device confirms conformity with the European requirements of the Medical Device Regulation (2017 / 745 / EU Medical Device Directive (MDR)).

Hereby, Sarabec Ltd declares that this device complies with all relevant UK Statutory Instruments. The full declarations of conformity for this device are available from:

Sarabec Ltd., 15 High Force Road, MIDDLESBROUGH TS2 1RH. UK.

Sarabec's EU authorised representative: HUMANTECHNIK GmbH, Germany.

Specifications

Working range: 10 to 30°C, optimum ambient temperature: 25°C

Operation: < 90%, non-condensing (transport, storage and operation)

Protection class: **IP 40**

Electrical safety: Protection class 2

Transmission frequency: 868,425 MHz

Max. transmission power: <10 mW

Transmission range: up to 200 m in open spaces, up to 80 m within buildings

Dimensions: 50 mm x ø 95,5 mm

Weight: 155 g (including mount)

Expected service life of the device: approx. 10 years

Compatible *Lisa-Signolux* receiver devices

The hazard detection devices are compatible with all receivers of the *Lisa-Signolux* product range:

- Receiver S (A-2616-W)
- Receiver T (A-2617-0)
- Portable vibrating receiver (A-2619-0)
- Receiver alarm clock (A-2634-0 / W)
- Flashing lamp (A-2644-W)
- Flashing lamp with Type F (CEE 7/3) socket (A-2645-0)
- Receiver T Connect (A-2676-0)

Certified accessories:

- Adhesive pad for Humanteknik hazard detection devices (A-2939-0)

All serious incidents must be reported to the manufacturer and the according local authority!

Technical specifications may be subject to change without prior notice.

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Humantechnik GHL AG