earis[®]
MAX

DE

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Seite 2

EN

Important notices

Page 6

FR

Avis importants

Page 12

NL

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Pagina 17

ES

Avisos importantes

Página 21

IT

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Pagina 25

SE

Viktiga anvisningar

Sida 29

FI

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Sivu 33

DK

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Side 37

PL

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Strona 41

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Important Safety Instructions:

Please observe the following important safety instructions before using your Bluetooth headphones. These instructions include general device safety guidelines as well as possibly specific information relevant to your model. It is important to heed the warnings and hazards to avoid injury or damage to the device.

Before using your headphones, please read all provided instructions carefully and thoroughly, especially these safety instructions.

Failure to follow these instructions may result in injury or damage.

Attention:

Your headphones can be set to a high volume. To protect your hearing or the hearing of individuals with normal hearing sensitivity, especially that of children, from hearing damage due to excessive volume, please observe the following guidelines:

- Set a low volume level before putting on the headphones.
- Adjust the volume to a comfortable level during use.
- If you have a pacemaker, ensure with your doctor or the pacemaker manufacturer that its functions are not affected before using this product.
- If you use other medical devices, only use the device while observing the safety instructions.
- The use of headphones can greatly reduce environmental auditory perception. This can lead to significant dangers, especially in traffic. It is extremely important that the user's hearing is not impaired to such an extent that traffic safety is endangered. Therefore, please absolutely avoid using headphones in traffic, especially near potentially dangerous machinery or work equipment.
- Furthermore, you should avoid using headphones in situations where unimpaired hearing is necessary, such as when crossing a railway crossing or on construction sites.

Additional Safety Instructions:

Storage of the User Manual:

Keep this user manual carefully stored so that it is accessible to other users at all times.

Damaged Device:

Do not use the device if it is obviously damaged.

Environmental Compatibility:

Use the device only in environments where 2.4 GHz Bluetooth® wireless technology is allowed.

Operating Environment:

The device and especially the power adapter are intended for operation in dry indoor environments only.

Temperature Requirements:

Proper function is ensured at room temperatures from 5°C to 35°C.

Safety for Children and Pets:

Handle the device and accessories carefully at all times and place them only in dust-free and clean locations.

Careful Handling:

Handle the device and accessories carefully at all times and place them only in dust-free and clean locations.

Magnets:

The device contains magnets. Avoid contact with credit cards, implanted medical devices, and other devices that may be affected by magnets. Maintain a distance of more than 15 cm from medical devices. If you suspect interference with your medical device, stop using the headphones and consult your doctor or the manufacturer of the medical device.

Power Cables and Outlets:

Do not use damaged power cables or plugs, and ensure that the power outlets are secure. Loose connections can cause electric shock or fire. Do not touch the device, power cables, plugs, or outlet with wet hands or any other wet body part as this can cause electric shock.

Cables and Chargers:

Do not pull the power cable tightly to remove it, and avoid bending or damaging the power cable to prevent electric shock or fire. Do not directly connect the positive and negative terminals of the charger as this can lead to fire and serious injury. Use only batteries, chargers, accessories, and consumables approved by the manufacturer to avoid damage to the device and ensure your safety..

Operating Environment:

Do not use the device outdoors during a thunderstorm to prevent electric shock or device malfunction. Do not carry the device in the back pocket or on the hip to prevent damage, explosions, or fires, and to avoid injury.

Shock Resistance and Disposal:

Avoid shocks or strong vibrations as this can cause damage to the device and battery, as well as malfunctions and a shortened lifespan. Handle the device and charger carefully and dispose of them properly according to local regulations to prevent environmental damage.

Temperature and Foreign Objects:

Protect the device, battery, and charger from extreme temperatures and avoid contact with conductive materials to prevent short circuits, overheating, or fires. Do not place foreign objects between the device and the inductive charger to ensure proper charging and prevent overheating.

Safety Precautions:

Do not use the device in areas with a high concentration of dust or particles, and do not store it there. Do not use a damaged or leaking lithium-ion battery (Li-Ion), and contact an authorized service center for safe disposal.

Please follow these safety guidelines carefully to avoid injuries and damage.

Correct disposal of this product (electrical and electronic waste)

The labeling on the product, accessories, or documentation indicates that the product and electronic accessories such as the charger, headset, and USB cable should not be disposed of with regular household waste. Please dispose of this device and its accessories separately from other waste to avoid harming the environment or human health through uncontrolled waste disposal.

Help ensure proper disposal of old devices and accessories to promote sustainable recycling of resources. Private users should contact the retailer where the product was purchased or contact the relevant authorities to find out where old devices or accessories can be dropped off for environmentally friendly disposal.

Commercial users should contact their supplier and follow the terms of the sales contract. This product and electronic accessories should not be disposed of together with other commercial waste.

This device complies with the requirements of the following EU directives:

- 2011/ 65/EU + 2015/863/EU RoHS-Directive
- 2012/19/EU WEEE-Directive
- 2014/53/EU RED-Directive

Compliance with the above directives is confirmed by the CE mark on the device. CE conformity declarations are available online at **www.humantechnik.com**.

This device complies with the requirements of the following IC directives: RSS247 / ICES-003

Hereby, Sarabec Ltd declares that this device complies with all relevant UK Statutory Instruments. The full declarations of conformity for this device are available from

Sarabec Ltd., 15 High Force Road,
MIDDLESBROUGH TS2 1RH. UK.
Sarabec's EU authorised representative:

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USA

Federal Communication Commission Interference Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC Caution: Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This device and its antenna(s) must not be co-located or operating in conjunction with any other antenna or transmitter.

FCC Declaration of Conformity (DoC)

FCC ID	U94MAX0124R
Model Name	HM-01 earisMAX

Federal Communication Commission (FCC) Radiation Exposure Statement: This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. End user must follow the specific operating instructions for satisfying RF exposure compliance.

HUMANTECHNIK

Model No: HM-01, earisMAX

We, Humantechnik GmbH, Im Wörth 25, DE-Weil am Rhein, Tel. +49 (0) 76 21/9 56 89-0, declare the above comply with the requirements of Federal Communications Commission. This device complies with Part 15 of FCC rules. Operation is subjected to the following conditions:
1) This device may not cause harmful interferences, and
2) This device must accept any interferences received that may cause undesired operation.

Responsible Party: Daniel Müller

Canada Industry Canada Statement

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RS-S(s). Operation is subject to the following two conditions:

1. This device may not cause interference.
2. This device must accept any interference, including interference that may cause undesired operation of the device.

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CAN ICES-3 (B)/NMB-3(B)

Radiation Exposure Statement:

The product comply with the Canada portable RF exposure limit set forth for an uncontrolled environment and are safe for intended operation as described in this manual. The further RF exposure reduction can be achieved if the product can be kept as far as possible from the user body or set the device to lower output power if such function is available.

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